
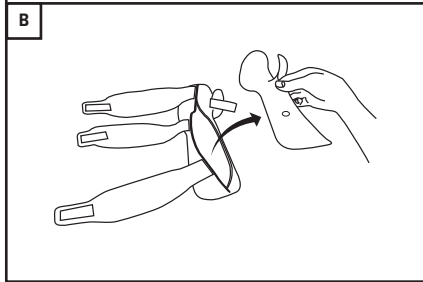
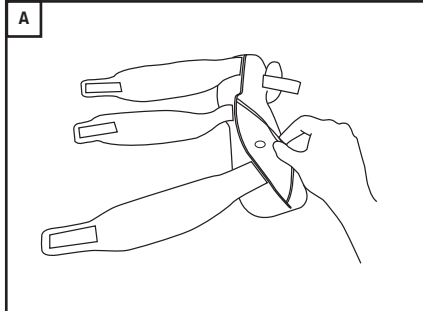
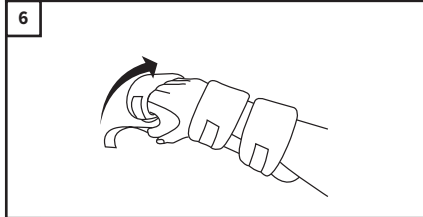
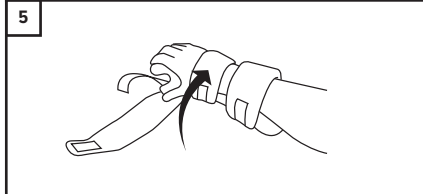
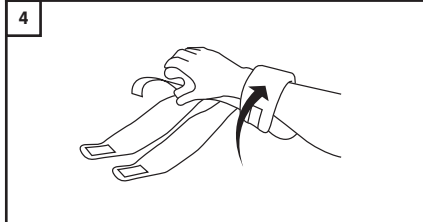
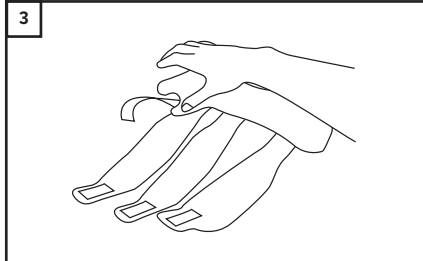
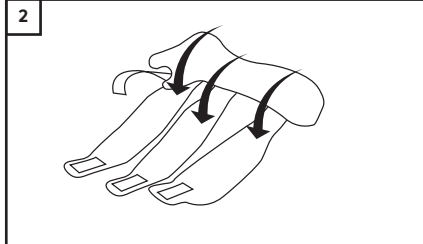
JRS - Functional Resting Splint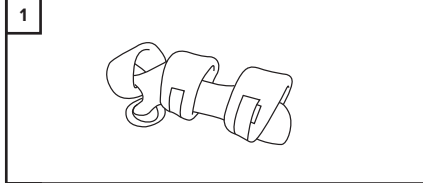

ENGLISH

Product Description

A functional resting splint that is available in three options: Padded, Easy-Liner and Thermoplastic Insert.

Usage Information (See diagram)

- Open all hook / loop fastenings.
- Position wrist, hand and thumb in brace as shown.
- Close hook / loop fastening.
- Re-adjust for comfort.
- Take care not to over tighten brace, re-adjust for comfort as required while ensuring brace is in correct position.

Sizes and Fit

Size	Wrist Circumference
XS	11.5 - 13 cm
S	14 - 15 cm
M	16.5 - 18 cm
L	20 - 21.5 cm
XL	23 - 24 cm

Indications for Use

Provides support to the wrist and hand for an array of injuries, conditions, pre-and post-operations such as:

- Arthritis.
- Neurological injuries and conditions .
- Soft tissue injuries and conditions.
- Post fracture management.

This is a good for patients requiring a brace on a longer-term basis and the fully padded cover offers comfort and compliance to those patients that are sensitive.

It is recommended that individuals seek guidance from a medical practitioner regarding the most appropriate brace and wearing schedule.

Contraindications

- Do not use over open wounds.
- Any known allergy and/ or hypersensitivity to any materials listed in the product composition.
- Perform regular skin and circulation checks.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

- We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.
- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Remove splint (See diagrams A & B).
- Close over all Hook/Loop straps.
- Hand wash in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check Hook/Loop fasteners are operating securely.

Product Composition

Polyester. Cotton, Nylon, Polyamide, Elastane.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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