

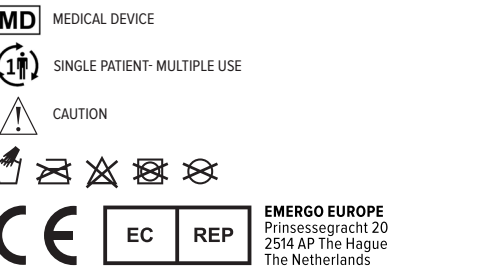
ASPEN® HINGED WRIST

WRIST / HAND ORTHOSIS
HANDGELENK-/HANDORTHESE
ÓRTEŠIS DEL POLSO E DELLA MANO
ORTHÈSE DU POIGNET / DE LA MAIN
POLS / HAND ORTHOSE
ORTÉZA NA ZÁPĚSTÍ/RUKU
HÄNDELE / HÄND ORTOSO
ΟΡΘΩΣΗ ΚΑΡΠΙΟΥ / ΧΕΙΡΙΟΥ
HÄNDELEDS/HÄNDORTOSE
ORTEZA NADGARSTKA / REKI
RANNE / KÄSIORTOSI
HANDLEDS/HANDORTOS
ОРТЕЗА ДЛЯ ЛУЧЕЗАПЯСТНОГО СУСТАВА
ОРТЗОЕ ДЕ ПУЛСО/МĂО
腕托/手托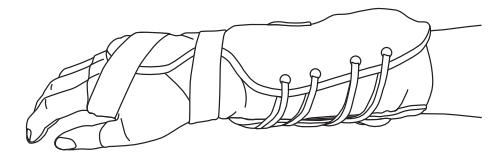

Aspen Medical Products, LLC
 6481 Oak Canyon, Irvine, CA 92618-5202 USA
 aspenmp.com • +1-800-295-2776 • +1-949-681-0200
 ©2021 Aspen Medical Products, LLC. All Rights Reserved.
 LB0039B revised 05/2021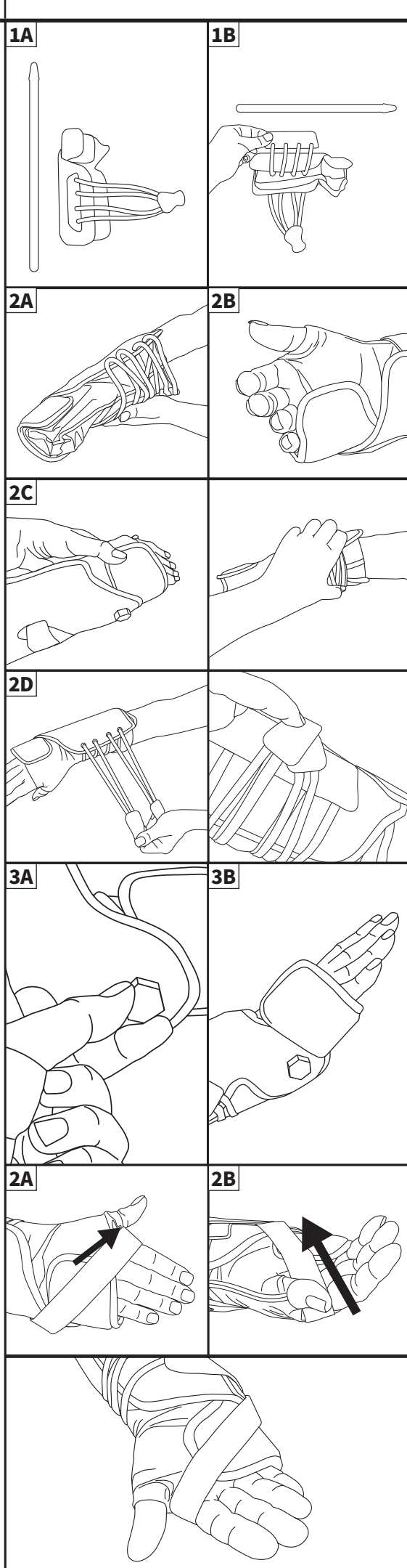

ENGLISH

1. PREPARATION
 A. Remove the Aspen Hinged Wrist from the packaging. Remove the support strap and detach the pull tabs from the brace.
 B. Detach the sizing strap and slide to adjust sizing. Reattach the sizing strap.

2. APPLICATION
 A. Carefully slide hand through brace sleeve.
 B. Ensure thumb slides through thumb hole at the top of the sleeve.
 C. Gently mold the aluminum frame to the patient's arm and hand.
 D. Using the pull tabs, carefully tighten the laces until appropriate compression is achieved. Secure the pull tabs to the brace.

3. HINGE ADJUSTMENT
 A. To adjust angle of the wrist, loosen the hinge dial.
 B. Adjust angle of the wrist brace forward or back to desired setting. Tighten the hinge dial.
 Note: Adjust setting before applying the brace to the patient.

4. SUPPORT STRAP
 A. Attach the support strap to the wrist side of the brace. Pull the strap across the back of the hand as shown.
 B. Wrap the strap between the thumb and forefinger and pull across palm of the hand. Secure the strap with the hook to the brace.
 Note: Adjust strap for best fit.

INTENDED USE
 To provide sagittal and coronal plane stabilization of the wrist.

INDICATIONS
 Conditions or injuries requiring stabilization of the wrist/hand in a prescribed position of flexion, neutral or extension.

CONTRAINDICATIONS
 Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

PRECAUTIONS
 READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

CARE INSTRUCTIONS
 Hand wash only at 30° C. Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

MATERIALS
 Acetal, Brass, Copper, Aluminum, Ethylene Vinyl Acetate, Stainless Steel, PVC, Nylon, Polyurethane Fiber (Lyra), Cotton/Polyurethane Fiber (Spandex), Polyethylene, Thermoplastic Elastomer.

NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL
 Product may be safely disposed in accordance with local laws.

REPORTING
 Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

LIMITED WARRANTY
 Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or parts).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
 (a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
 (b) The product is altered;
 (c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
 (d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OF USE. AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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Společnost Aspen Medical Products, LLC nebuduje mít podle této omezené záruky žádnou povinnost v případě, že:
(c) Do produktu jsou vloženy jakékoli součásti, které nedodává společnost Aspen Medical Products, LLC; nebo
(d) Produkt se nepoužívá v souladu s pokyny k použití společnosti Aspen Medical Products, LLC.