# EXTENSION / FLEXION STOP FRAME & KEY

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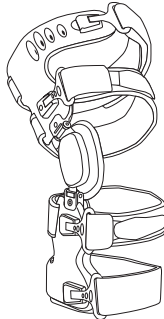
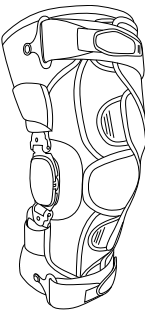
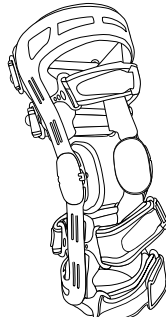
CAUTION

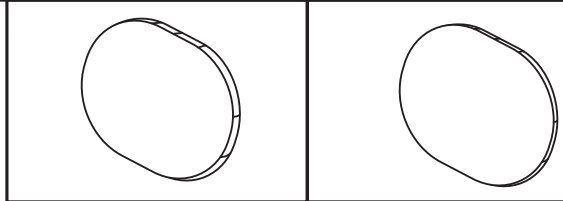
EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands

Aspen Medical Products, LLC 6481 Oak Canyon, Irvine, CA 92618-5202 USA aspenmp.com • +1-800-295-2776 • +1-949-681-0200


©2021 Aspen Medical Products, LLC. All Rights Reserved.

LB0050C revised 06/2021


THICK CENTER PAD DICKES MITTLERES POLSTER ALMOHADILLA CENTRAL GRUESA IMBOTTITURA CENTRALE SPESSA COUSSINET CENTRAL ÉPAIS DIKKE MIDDELSTE PAD SILNÁ STŘEDOVÁ PODLOŽKA ТУК МІДТЕРПУДЕ ПАХУ КЕНТРИКО МАΞΙΛΑΡΑΚΙ ΤΥΚΚ SENTERPLATE GRUBA PODKLADKA ŚRODKOWA PAKSU KESKIPHEMUSTE TJOCK MITTDYNA ТОЛСТАЯ ЦЕНТРАЛЬНАЯ НАКЛАДКА ALMOFADA CENTRAL ESPESA 厚中心垫

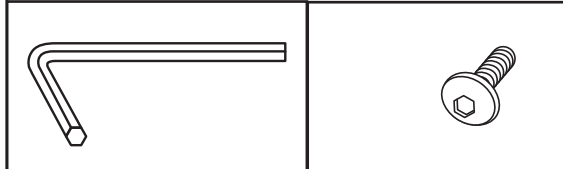THIN CENTER PAD DÜNNES MITTLERES POLSTER ALMOHADILLA CENTRAL FINA IMBOTTITURA CENTRALE SOTTILE COUSSINET CENTRAL FIN DUNNE MIDDELSTE PAD TENKÁ STŘEDOVÁ PODLOŽKA TYND MIDTERPUDE ΛΕΠΤΟ ΚΕΝΤΡΙΚΟ ΜΑΞΙΛΑΡΑΚΙ ΤΥΝΝ SENTERPLATE CIENKA PODKLADKA ŚRODKOWA OHUT KESKIPHEMUSTE TUNN MITTDYNA ΤΟΝΚΑЯ ЦЕНТРАЛЬНАЯ НАКЛАДКА ALMOFADA CENTRAL FINA 薄中心垫


ALLEN WRENCH, 2MM INBUSSCHLÜSSEL, 2MM LLAVE ALLEN, 2 MM CHIAVE A BRUGOLA, 2 MM CLÉ ALLEN, 2MM INBUSSLEUTEL, 2MM IMBUSOVÝ KLÍČ, 2 MM UNBRAKONØGLE, 2MM ΚΛΕΙΔΙ ALLEN, 2XΛ UNBRAKONØKKE, 2 MM KLUCZ IMBUSOWY, 2 MM KUUSIOKOLORUUVI, 2MM INSEKXYCKEL, 2MM КЛЮЧ ALLEN, 2 MM ALLEN WRENCH, 2MM 内六角扳手, 2毫米

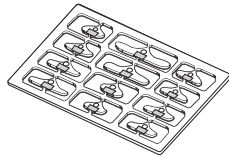ALLEN SCREW, 3MM INBUSSCHRAUBE, 3MM TORNILLO ALLEN, 3 MM CHIAVE A BRUGOLA, 3 MM VIS ALLEN, 3MM INBUSSLEUTEL, 3MM IMBUSOVÝ ŠROUB, 3 MM UNBRAKOSKRUE, 3MM ΒΙΔΑ ALLEN, 3XΛ UNBRAKOSKRUE, 3 MM SRUBA IMBUSOWA, 3 MM KUUSIOKOLORUUVI, 3MM INSEKXYCKEL, 3MM ВИHT ALLEN, 3MM PARAFUSO ALLEN, 3MM 内六角螺丝, 3毫米

PLASTIC ANGLE ADJUSTMENT SET KUNSTSTOFFWINKEL-EINSTELLSATZ JUEGO DE AJUSTE DE ÁNGULO DE PLÁSTICO SET DI REGOLAZIONE DELL'ANGOLO IN PLASTICA JEU DE RÉGLAGE D'ANGLE EN PLASTIQUE KUNSTSTOF HOEKINSTELSET PLASTOVÁ SADA PRO NASTAVENÍ ÚHLU JUSTERINGSSET FOR PLASTVINKEL ΠΛΑΣΤΙΚΟ ΣΕΤ ΠΥΘΜΙΣΗΣ ΓΩΝΙΑΣ VINKELJUSTERINGSSET I PLAST ZESTAW PLASTIKOWYCH CZĘŚCI DO USTAWIANIA NOGI POD KAŃEM MUOVINEN KULMANSÄÄTÖSARJA PLASTVINKELJUSTERINGSATS ПЛАСТИКОВЫЙ НАБОР ДЛЯ РЕГУЛИРОВКИ УГЛА CONJUNTO DE AJUSTE DE ÁNGULO DE PLÁSTICO 塑料角度调整套件

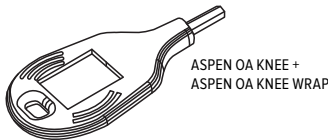VARUS / VALGUS ADJUSTABLE KEY EINSTELLBARER VARUS-VALGUS-SCHLÜSSEL LLAVE DE AJUSTE DE VARO Y VALGO CHIAVE REGOLABILE VARO/VALGO CLÉ RÉGLABLE VARUS / VALGUS VARISERENDE/ VALGISERENDE VERSTELBARE SLEUTEL NASTAVITELNÝ KLÍČ VARUS / VALGUS VARUS / VALGUS JUSTERBAR NØGLE PYÖMIZOMENO ΚΛΕΙΔΙ VARUS / VALGUS VARUS/VALGUS JUSTERBAR NØKKE KLUCZ NASTAWNY ZMIENIAJĄCY USTAWIENIE DO SZPOTAWOŚCI / KOŚLAWOŚCI VARUS/VALGUS SÄÄDETTÄVÄ AVAIN VARUS / VALGUS JUSTERBAR NYCKEL РЕГУЛИРУЕМЫЙ КЛЮЧ VARUS/VALGUS CHAVE AJUSTAVEL VARUS/VALGUS 内翻/外翻可调按钮

## ENGLISH

### INTENDED USE

Not applicable.

### INDICATIONS

Not applicable.

### CONTRAINDICATIONS

Hipersensibilidad o alergias to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

### PRECAUTIONS

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

### CARE INSTRUCTIONS

Hand wash only at 30° C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

### MATERIALS

Nylon, EVA, Steel.

### NOT MADE WITH NATURAL RUBBER LATEX.

### DISPOSAL

Product may be safely disposed in accordance with local laws.

### REPORTING

Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

### LIMITED WARRANTY

Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:

- (a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES, THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location.

A listing of authorized distributors in your area is available upon request.

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Dans les cas suivants, Aspen Medical Products, LLC n'aura aucune obligation au titre de cette garantie limitée:
- (a) Le produit n'a pas été acheté auprès d'Aspen Medical Products, LLC ou via ses canaux de distribution autorisés;
(c) Toute pièce non fournie par Aspen Medical Products, LLC a été insérée dans le produit; et
(d) Le produit n'est pas utilisé conformément au mode d'emploi d'Aspen Medical Products, LLC.