

LUMBAR SACRAL ORTHOSIS
LUMBAL-SAKRALE ORTHESE
ORTESIS LUMBO-SACRALE
ORTHÈSE LOMBO-SACRÉE
LUMBALE SACRALE ORTHÈSE
BEDERNÍ KRÍŽOVÁ ORTÉZA
LUMBAL SAKRAL ORTOSIS
ΟΡΘΗΣΗ ΛΟΜΒΟ-ΣΑΚΡΑΛΕ
LUMBAL KORSRYGGORTROE
ORTEZA ŁĘDŹOWO-KRZYŻOWA
LANNERANGAN SAKRAALINEN ORTOOSI
LUMBAL SAKRAL ORTOS
ОРТЕЗ ДЛЯ ПОЯСНИЧНО-КРЕСТЦОВОГО ОТДЕЛА
ПОЗВОНОЧНИКА
ORTÈSE SAKRAL LOMBAR
腰骶矫形器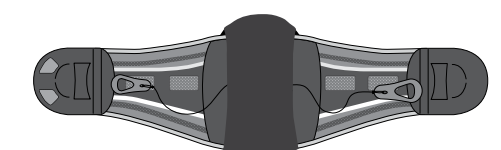

1
2
3A
3B
3C
3D
4A
4B

MD MEDICAL DEVICE
1 SINGLE PATIENT - MULTIPLE USE
CAUTION
CE EC REP
EMERGO EUROPE
Westervoorsteed 60,
6827 AT Arnhem
The Netherlands
Pat. aspenmp.com/patents

Aspen Medical Products, LLC
6481 Oak Canyon, Irvine, CA 92618-5202 USA
aspenmp.com • +1-800-295-2776 • +1-949-681-0200
©2026 Aspen Medical Products, LLC. All Rights Reserved.
LB0059E revised 04/2026

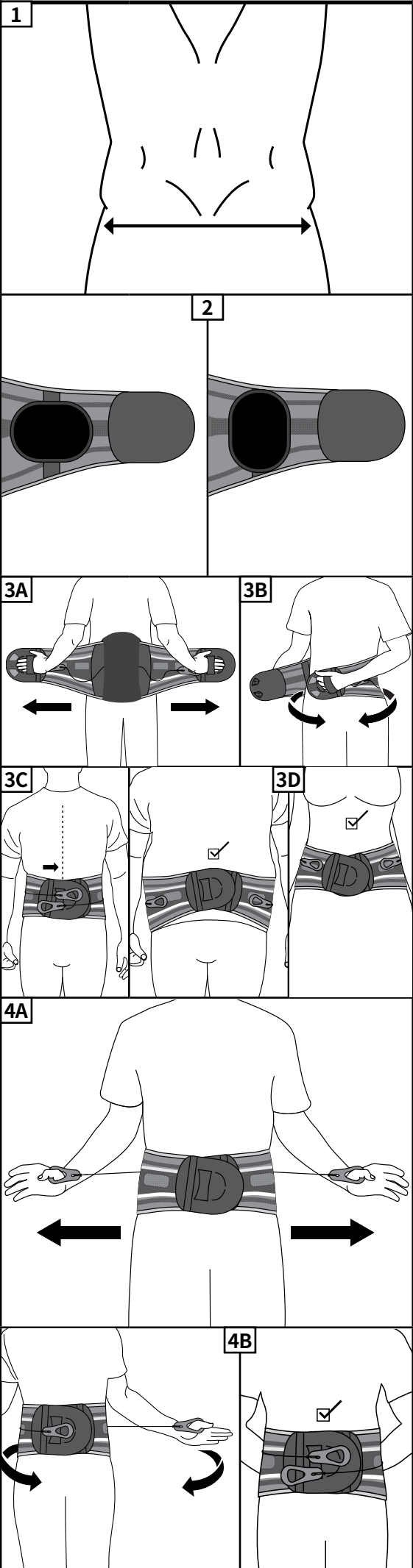ENGLISH
1. PREPARATION
Select appropriate brace size by measuring around the waist and hips.
2. ADJUST LUMBAR PAD
With brace extended fully, place one lateral pad on midline on each side panel of the brace. Press down to secure to brace.
3. PLACEMENT
A. Apply the fully stretched brace making sure the back panel is centered and low on the back.
B. Overlap the ends of the brace so the hook material grips the opposite side loop as tightly as possible.
C. Make sure that the upper and lower parts of the orthosis are firmly seated and that the anterior panel is centered at the patient's midline.
D. Be sure the top and bottom of the brace is snug. If needed, angle ends to create a snug fit.
4. TIGHTEN & ADJUST
A. Slip your thumbs through the holes in the pull tabs. Pull away from your body. The left pull tab controls lower compression and the right pull tab controls upper compression. Adjust compression as needed.
B. When desired compression is achieved, place the hook side of the pull tabs down onto the loop material to secure.
INTENDED USE
To provide motion restriction and stability of the lumbar-sacral spine.
INDICATIONS
Postoperative support, Deconditioned trunk musculature, Post laminectomy syndrome, Surgical Decompression procedures, Non-surgical Decompression procedures, Degenerative disc disease, Bulging or herniated disc, Non-specific low back pain, Chronic low back pain, Spondylolysis, Spondylolisthesis, Spinal stenosis, Ligament sprains / Muscle strains, Spondylolysis (Osteoarthritis/Degenerative Joint Disease), Facet syndrome, SI dysfunction, Sacroiliitis, Radiculopathy, Injections, Radiofrequency Ablation, Lumbar Multitidus Muscle Atrophy.
CONTRAINDICATIONS
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.
PRECAUTIONS
READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.
CARE INSTRUCTIONS
Hand wash only at 30° C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.
MATERIALS
Nylon, Polyester, Polyethylene, Polyurethane, Acetal, PVC, ABS.
NOT MADE WITH NATURAL RUBBER LATEX.
DISPOSAL
Product may be safely disposed in accordance with local laws.
REPORTING
Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).
LIMITED WARRANTY
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or part(s). Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use. THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THE OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.
This warranty gives you specific legal rights and you may have other rights which vary from location to location.
A listing of authorized distributors in your area is available upon request.

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Aspen Medical Products, LLC non avrà alcun obbligo ai sensi della presente garanzia limitata nei seguenti casi:
(c) Il prodotto non è stato inserito parti non fornite da Aspen Medical Products, LLC o
(d) Il prodotto non è utilizzato ai sensi delle istruzioni per l'uso fornite da Aspen Medical Products, LLC.